

Borreliella spp. Antigens

Lyme disease is the most common tick-borne disease in the northern hemisphere, caused by *Borreliella spp*. Even though Lyme disease was originally identified in the town of Lyme, Connecticut (USA) in the mid 1970s, prospective studies revealed an earlier history in untreated European patients (Borchers *et al.* 2015). The various clinical manifestations include flu-like symptoms and a characteristic circular skin rash called erythema migrans (EM).

Even though more abundant in the United States the total number of annual cases of Lyme Disease in Europe is estimated to be approximately 3-fold higher than the number of reported cases. Climate changes with a subsequent expansion of vector tick territory, along with changes in land use might affect changes in epidemiology of Lyme Disease and other tick-borne infections in the future (Andersen and Davis 2016).

The complex endozoonotic cycle of *Borreliella spp*. in Europe usually involves transmission to a broad range of vertebrates by infected ticks of the species *Ixodes ricinus* (Rizzoli *et al.* 2011). Due to the presence of different ecotypes within the species and utilization of different

Ordering Information		
41100 41101	Borreliella afzelii BmpA	0.1 mg 1.0 mg
40900 40901	Borreliella afzelii DbpA	0.1 mg 1.0 mg
41000 41001	Borreliella afzelii OspA	0.1 mg 1.0 mg
41800 41801	Borreliella afzelii OspC	0.1 mg 1.0 mg
42300 42301	Borreliella afzelii p100	0.1 mg 1.0 mg
42700 42701	Borreliella bavariensis DbpA	0.1 mg 1.0 mg
40700 40701	Borreliella bavariensis p58	0.1 mg 1.0 mg
41400 41401	Borreliella bavariensis VIsE1	0.1 mg 1.0 mg
42800 42801	Borreliella garinii DbpA	0.1 mg 1.0 mg
41900 41901	Borreliella garinii OspC	0.1 mg 1.0 mg
42900 42901	Borreliella spielmanii DbpA	0.1 mg 1.0 mg
40800 40801	Borreliella spielmanii OspC	0.1 mg 1.0 mg

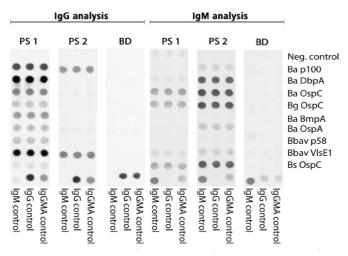


Figure: Immunodot analyses of positive (PS1, PS2) and negative (BD) samples for Borreliella afzelii (Ba), B. bavariensis (Bbav), B. garinii (Bg), B. spielmanii (Bs). The presence of IgG (left) and IgM (right) antibodies was determined spotting triplicates of recombinant DIARECT antigens derived from Borreliella spp. on nitrocellulose membrane.

host reservoirs, Margos et al. (2009) analyzed intra- and interspecific relationships by multilocus sequence analysis (MLSA). Apart from B. garinii and B. bavariensis, cases in Europe can be caused by B. afzelii, B. spielmanii, and B. burgdorferi sensu stricto. Similar immunogenic proteins of Borreliella species are associated in US and European cases. Especially outer surface proteins A and C (OspA, OspC), and variable major protein-like sequence expressed 1 (VIsE1) appear to be important for infection and immune evasion. Basic membrane protein A (BmpA) serves as adhesin for binding to the host cell. Further immunogens are p58 and p100, whose cellular functions and pathogenic implications have not been clarified yet. The most sensitive protein for IgG antibody detection in all stages of Lyme disease was found to be VIsE1 followed by p100 and p58 (Goettner et al. 2005).

The DIARECT's recombinant *Borreliella spp.* antigens are expressed in either *E. coli* or the baculovirus/insect cell expression system.

References:

Andersen and Davis (2016) Int J Dermatol. 33: 2358-2365 Borchers *et al.* (2015) J Autoimmun. 57: 82-115 Goettner *et al.* (2005) J Clinical Microbio. 43: 3602-3609 Margos *et al.* (2009) Appl Environ Microb. 75: 5410-5416 Rizzoli *et al.* (2011) Euro Surveill. 16: pii 19906

In some countries the use of certain antigens in diagnostic tests may be protected by patents. DIARECT is not responsible for the determination of these issues and suggests clarification prior to use.

210414 Rev05

